

MediSure®

Adjustable Lancing Device With Ejector

1. Wash your hands with soap and warm water and dry them thoroughly. Remove the lancing device cover.
2. Place a new lancet securely into the lancet carrier.
3. Remove the protective cap from the lancet. Do not discard this cap.
4. Place the cover back onto the lancing device.
5. Choose the desired depth of penetration to get a good blood sample with the least amount of discomfort. Adjust the depth setting by turning the cover on the lancing device.

Settings:

- 1 or 2 - for delicate skin.
 - 3 or 4 - for normal skin.
 - 5 or 6 - for thick or callused skin.
6. Hold the lancing device in one hand. Pull the sliding barrel forward with your other hand until an audible click is heard. The lancing device is now loaded and ready to use. The device may be returned to the original position by releasing the sliding barrel.
 7. To obtain a better sample of blood, lower the hand from which a sample is to be obtained and shake it a few times. Rub from the palm upward to the fingers with your other hand.
 8. Place the end of the lancing device against the finger from which the blood sample is to be obtained. Push the trigger button to puncture the skin. Set the lancing device aside and proceed with your blood glucose test according to blood glucose monitor's directions.
 9. Remove the adjustable comfort tip after the penetration has been made. Put the protective cover back onto the lancet in order not to expose the needle tip.
 10. Push forward the ejector, and dispose off the lancet in an approved container. Revert the ejector back; depress the trigger, so the lancing device will turn back to its original state. Put back on the adjustable comfort tip.

Note

- *The blood-borne pathogen (e.g. HBV, HCV, or HIV) may be transmitted if there is shared using of a medical device.*
- *The lancing device is intended to be used by a single person and should not be shared.*
- *You should wash hands thoroughly with soap and water after handling the meter, lancing device, or test strips.*
- *Please refer to the section of "How to Clean and Disinfect Your Device" for more detailed information on Cleaning and Disinfection.*



Manufactured by / fabriqué par

MediSure Canada Inc.

Suite 160
31A - 328 Speedvale Ave East
Guelph, Ontario, Canada N1E0J4
www.medisure.ca
1-855-MDI-SURE (855-634-7873)

CE0197

