



1. About the system

Medi+Sure® Multi Blood Glucose Meter (Front) Medi+Sure® Multi Blood Glucose Meter (Back)

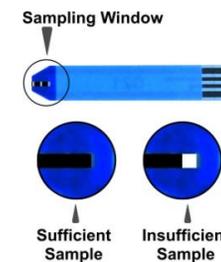


2. Install / Replace the Battery

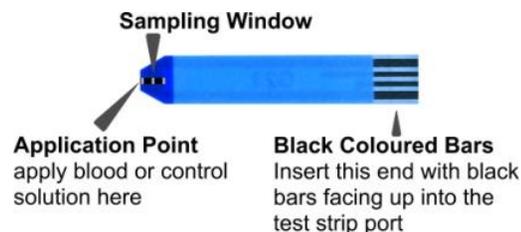

1. Open the battery cover by pressing on it and sliding it in the direction of the arrows on the battery cover.
2. If replacing battery, remove the old battery.
3. Insert a new battery firmly with the positive (+) side up.
4. Replace the battery cover. The LCD will display "Set."
5. Reset the date, time.

3. Set the Year, Date, Time, and Temperature Units

1. Press the "S" button (setting button) ≥ 3 seconds.
2. You will hear a beep, and the meter will power on. The LCD display shows date, time, temperature unit, thermometer symbol on the screen
3. Press the "S" button years flashes on the screen
4. Press the ">" button (right arrow button) or the "<" (left arrow button) until the correct years is displayed
5. Press the "S" button again, the month flashes on the LCD Screen.
6. Repeat step 4 and 5 to change the date, time and temperature unit you want to display.

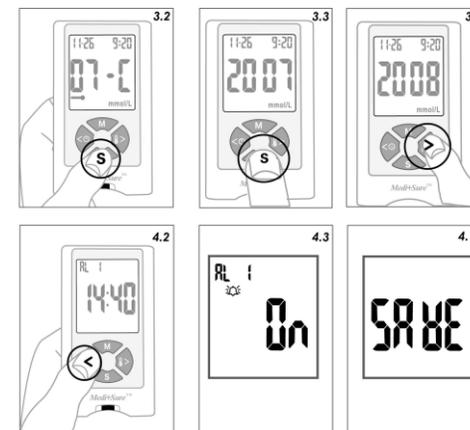


4. Alarm Setting

1. Press the "<" button ≥ 3 seconds on the front of the meter.
2. You will hear a beep, and the meter will power on. The LCD display shows AL 1(Alarm 1), and time on the screen.
3. Press the "S" button (setting button) to activate the alarm and alarm symbol flashes on the screen. You can set the alarm ON or OFF by pressing the ">" or "<" button.
4. Press the "S" button again, the hour flashed on the LCD screen.
5. Press the ">" button or "<" button until the correct hour is displayed.
6. Press the "S" button. The minute flashes.
7. Press the ">" button or "<" button until the correct minutes is displayed.
8. Press the "S" button, the meter beeps. The LCD display show "ok". The AL1 (Alarm 1) be setting finish.
9. Press the "S" button. You can set AL 2~4.
10. Press the "S" button > 3 seconds at any step the meter beeps. After "SAVE" displayed on the screen, the meter shouted off. All settings will be saved.

5. Prepare the single use lancing device and Blood Sampling

1. Wash your hands in warm, soapy water. Rinse well and **DRY** them thoroughly. You can also use an alcohol pad to clean your fingertip. It is necessary to make sure the site is **DRY** before testing..
2. Carefully pull off the protective cap
3. Push the single use lancing device firmly onto the chosen site to fire.
4. Dispose of device PA in a suitable sharps container

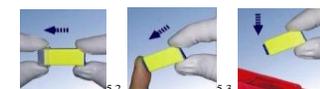

6. Testing Your Blood Glucose

1. The **Medi+Sure™** multi Blood Glucose Monitoring System is intended for use in the quantitative measurement of glucose in fresh capillary whole blood from the finger. It may be used for multiple patients in a professional healthcare setting.
2. Prepare the single use lancing device according to the lancing device instruction.
3. Take one test strip out of the test strip vial. Replace the vial cap immediately and close it tightly.
4. Insert this test strip (with the black bars facing up) into the test strip slot of the meter. The meter turns on automatically, and the code number appears on the screen. Make sure this number matches the code on the test strip vial.
5. When the blood drop symbol flashes on the meter, it means that you can start testing. You have 3 minutes to apply the blood sample. If you do not apply the sample within 3 minutes, the meter will turn off automatically.
6. Please read the device Instruction for Use for system information and instructions for use of the device before start.
7. Place the lancing device against your fingertip and press the trigger. To help blood drop form, lower your hand below your twist and gently massaging finger from palm to fingertip. Gently squeeze your fingertip until you get a drop of blood.
8. Pick up the meter and use the most comfortable way to touch the blood drop on the edge of the application point of the test strip. The blood will be drawn in to the test strip automatically. You will hear a beep letting you know the test has begun.
9. When you hear a beep, four dashed (- - -) will appear on the screen. Then the meter displays "- - -", "- -" and "-." in sequence. After countdown, the test results is displayed with date and time and stored in the meter's memory automatically.
10. Record the result in your Log Book.
11. Remove the test strip from the meter. Discard the used test strip and lancet in a puncture-proof container.
12. **The meter should be wiped with suitable disinfectant by following the section of "How to Clean Your Device" in User's Guide after using or before using in next patient.**