
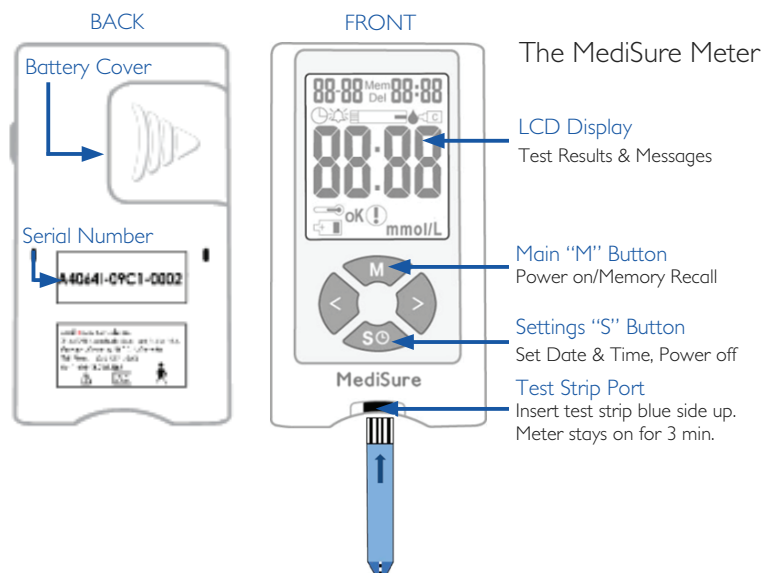
Multi User Meter (DG001MM-MULTI) QUICK START GUIDE

This Blood Glucose Monitoring System has been Licensed by Health Canada for Multi Patient Use. It is intended for use by health care professionals on multiple patients. It is certified as safe to use for over 18,000 tests and in-between cleaning protocols.


BATTERY INSTALLATION

1. Press down Battery Cover and slide right.
2. Firmly insert battery with positive (+) side up.

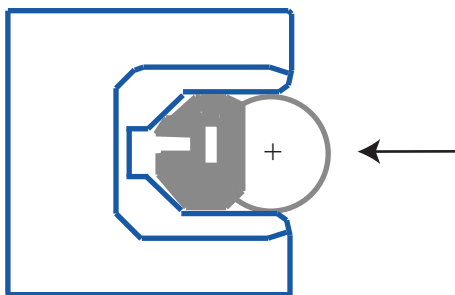SET DATE AND TIME


1. Press and hold the "S" button for 3+ seconds
2. Year flashes. Press the ">" or "<" buttons until the correct year is displayed.
3. Press the "S" button. Month flashes in the top left. Press the ">" or "<" buttons until the correct year is displayed.
4. Repeat Step 3 for the Day, Hour (24 hours) and minutes.
5. Press the "S" button. Meter will display "SET" and turn off. Date and time is now set.

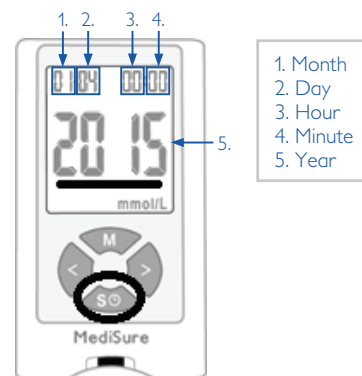
BEGIN TESTING

Wash hands with warm water and dry.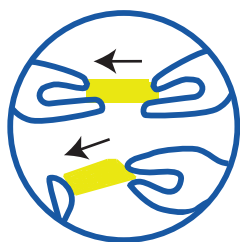


Put on Gloves and Prepare Single Use, Auto Disabling Lancet

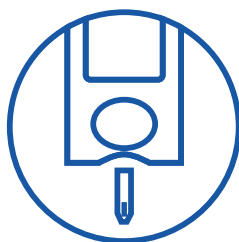Insert MediSure test strips in meter.

Apply blood sample. Results in 6 seconds!

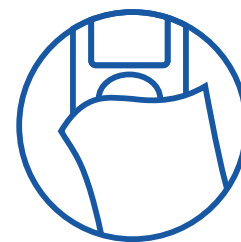Perform In Between Cleaning Protocol after each use

VIEW TEST RESULTS

1. Press the "M" button to turn on the meter. Press the "M" button again.
2. Last test results are displayed. Press "<" and ">" button to toggle results.
3. Press "M" button a third time to see 7, 14, 21, and 28 day averages.
4. Press "S" button to turn off Meter

Follow usage Instructions carefully. Use only with an auto-disabling, single use lancing device. A new pair of clean gloves should be worn by the user before testing each patient.. Carefully follow cleaning and disinfection protocols between each use

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