


QUICK START GUIDE


BATTERY INSTALLATION

1. Press down Battery Cover and slide right.
2. Firmly insert battery with positive (+) side up.

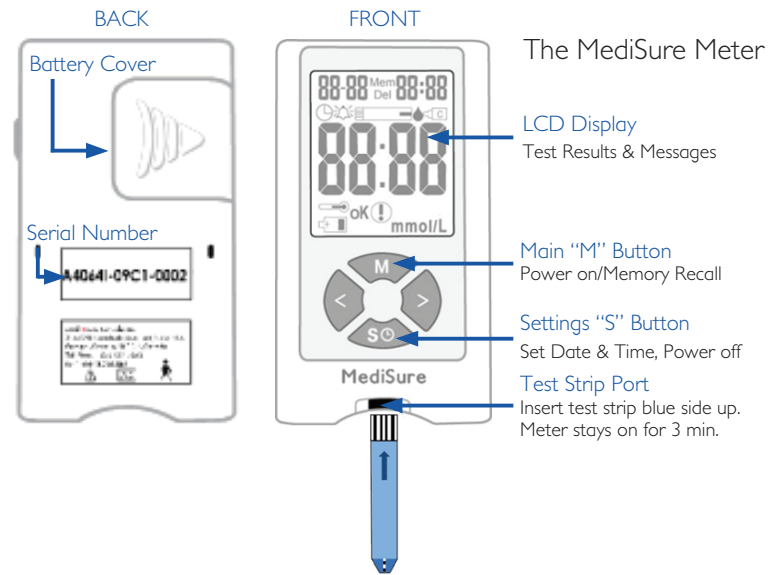

SET DATE AND TIME

1. Press and hold the "S" button for 3+ seconds
2. Year flashes. Press the ">" or "<" buttons until the correct year is displayed.
3. Press the "S" button. Month flashes in the top left. Press the ">" or "<" buttons until the correct year is displayed.
4. Repeat Step 3 for the Day, Hour (24 hours) and minutes.
5. Press the "S" button. Meter will display "SET" and turn off. Date and time is now set.

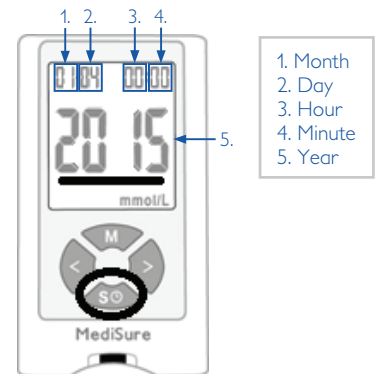


BEGIN TESTING



Wash hands with warm water and dry.



Install lancet into lancing device.

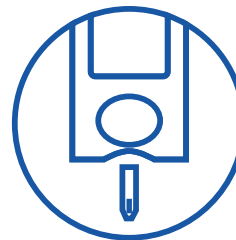

Insert MediSure test strips in meter.



Apply blood sample. Results in 6 seconds!

VIEW TEST RESULTS

1. Press the "M" button to turn on the meter. Press the "M" button again.
2. Last test results are displayed. Press "<" and ">" button to toggle results.
3. Press "M" button a third time to see 7, 14, 21, and 28 day averages.
4. Press "S" button to turn off Meter

